

## Instructions: VIT Applicator


**Please keep the Wings warm or heat them up to +40°C (+104°F) before inserting the VIT into the applicator!**

**Cold wings can break when pressing the wings together.**

VECTRONIC Aerospace has expanded the EAZI-BREED applicator with an ivory-colored adapter so that it can be used to insert the VERTEX VITs. The length of the adapter is 50mm and is glued to the applicator with a drop of superglue.

Please note that the VIT is available with three different wing spans:  
S (68 mm) M (88 mm) L (150 mm)

Please squeeze the wings of the VIT and insert it into the applicator. It is essential to ensure that the VIT's antenna runs in the opening of the adapter on the outside.

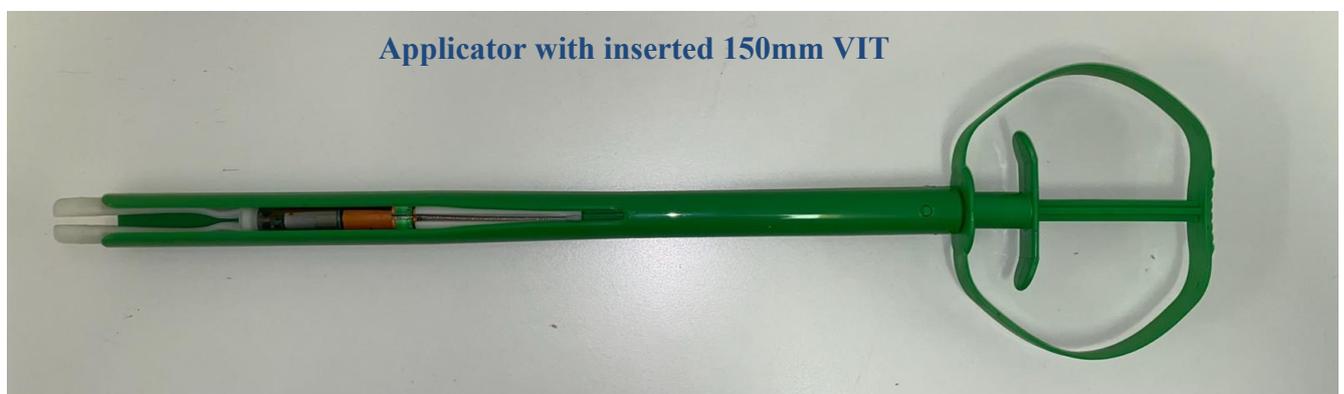


## Instructions: VIT Applicator

When you slowly start depressing the plunger of the applicator, the VIT is slowly ejected.


68mm VIT



88mm VIT

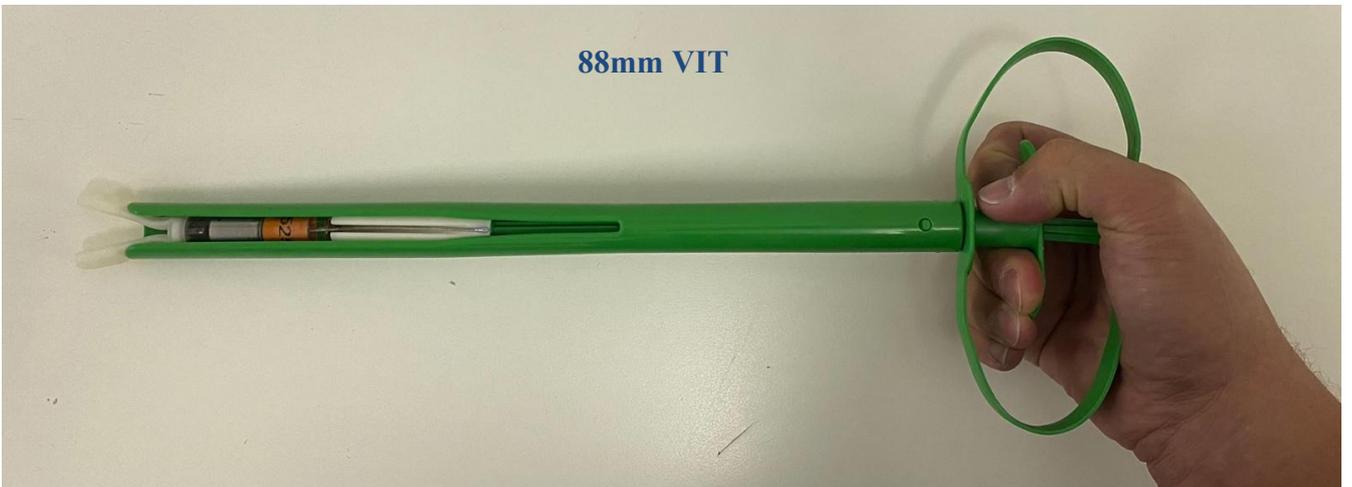


150mm VIT


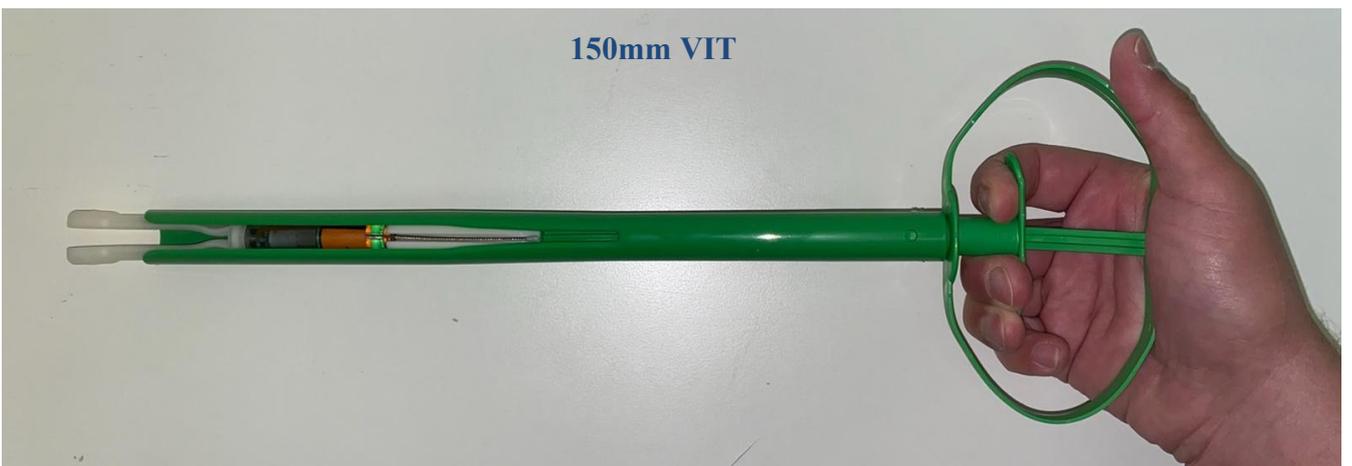


## Instructions: VIT Applicator

When you fully squeeze the applicator plunger, the VIT is ejected.

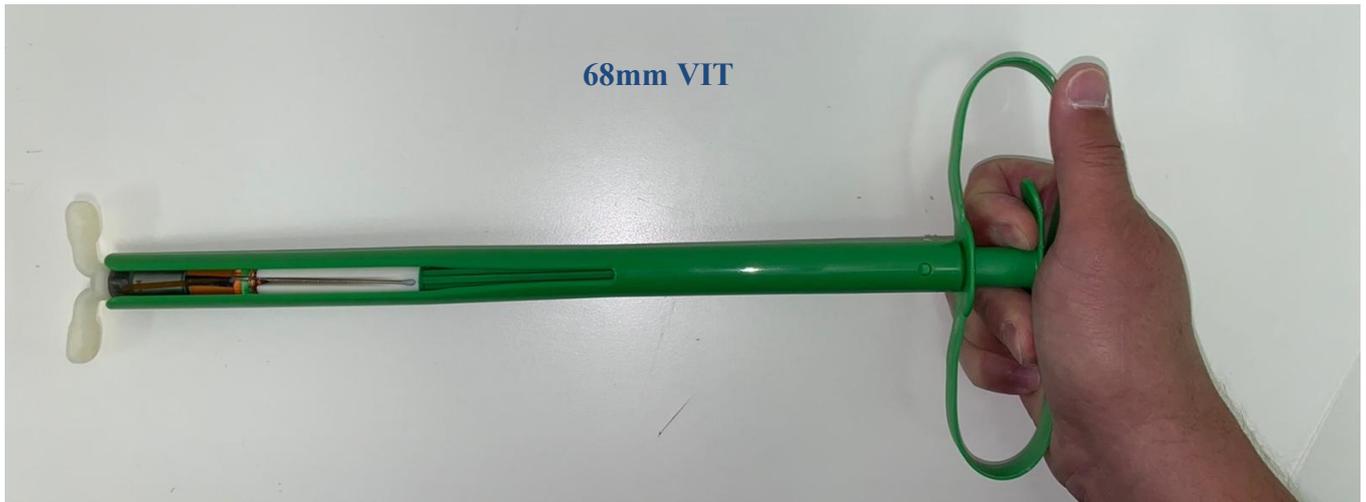

## Instructions: VIT Applicator

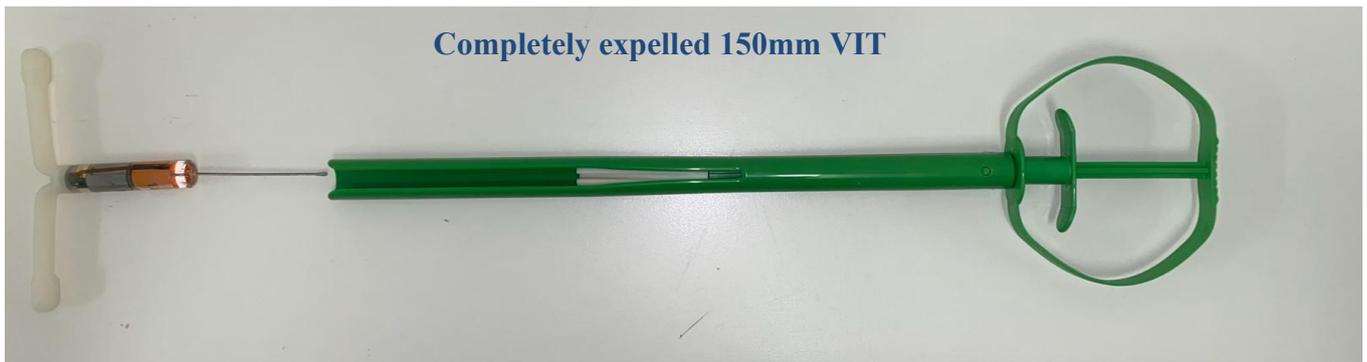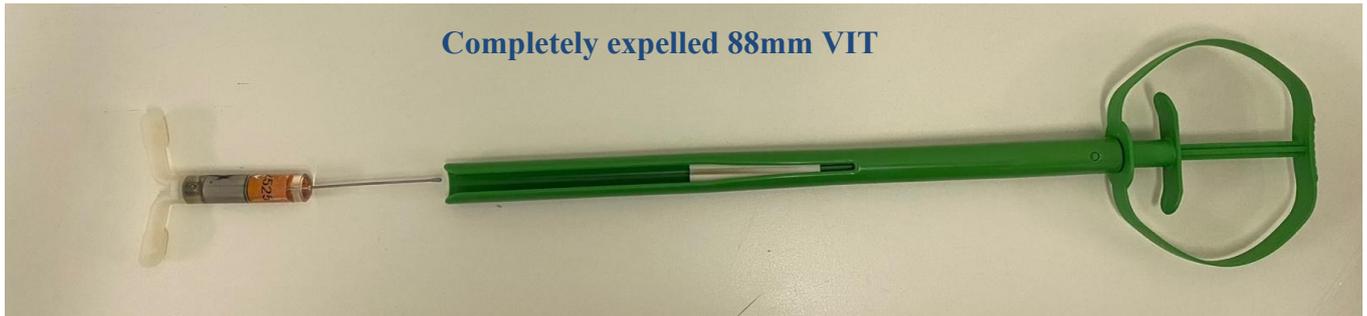

**Please keep the Wings warm or heat them up to +40°C (+104°F) before inserting the VIT into the applicator!  
Cold wings can break when pressing the wings together.**

**Optional: The original piston can be replaced with a VECTRONIC designed piston to ensure reliability when inserting a VIT. The additional piston is packed as a spare part.**



## How to Use the EAZI-BREED™ CIDR® Cattle Insert

For best results, follow these simple steps:

1. Wear protective gloves whenever handling the EAZI-BREED CIDR Insert.
2. Prepare a container of clean water with disinfectant solution. Wash the applicator with the water between uses.
3. Fit the body of the insert into the applicator with the tail along the slot. The two wings will be pushed together, protruding about one inch above the top of the applicator.
4. Apply a generous amount of lubricant to the tip of the insert.
5. Shift the animal's tail to one side, and clean the vulva.
6. Make sure the tail of the EAZI-BREED CIDR Insert is on the underside of the applicator, curling down, to ensure that the tail will be hidden from curious penmates.
7. Open the lips of the vulva and insert the applicator at a slightly upward angle, moving forward over the pelvic bone until it meets resistance.
8. Dispense the insert from the applicator by depressing the plunger, then slowly withdraw the applicator.
9. To prevent removal by curious penmates, you may want to clip the tail of the insert so that 2-1/2" protrude from the vulva.
10. To withdraw the insert seven days later, simply give the tail a gentle but firm pull to release the insert.
11. Dispose of used inserts in a sealed, plastic container in accordance with applicable local, state and federal regulations.



Breed More Cattle in Less Time